

Instructions

1. Cierre la faja ajustando firmemente las puntas de fijación Hook & Loop. Secure the belt by fastening the Hook & Loop closure

2. Coloque los tirantes y ajústelos deslizando la hebilla. Put on the suspenders and adjust them by sliding the clips

3. Estire los laterales elásticos exteriores. Stretch out the outer elastic sides

4. Asegúrese que estén firmemente ajustados. Make sure they are securely tightened

Hebillas ajustables de suave deslizamiento. Smooth glide adjustable clips. Laterales elásticos con cintas reflejantes de alta visibilidad. Elastic sides with high-vis panels reflective stripes



Cinturón de soporte abdominal de fijación Hook & Loop. Abdominal support panel with Hook & Loop closure



4 varillas / 4 stays. Reduce el riesgo de lesiones y molestias al cargar objetos pesados. Minimize injuries from weight lifting

Tirantes elásticos. Elastic suspenders

Previene ciertas deformidades corporales / Prevents some body deformities. Composición: 40% poliéster, 30% elastano, 20% nylon, 5% polietileno, 5% otros.

Instrucciones de cuidado: lavar a mano con agua fría y detergente. No usar blanqueador ni máquina secadora. Lavar por separado. Secado horizontal y a la sombra. No exprimir. No planchar.

ATENCIÓN

La faja es preventiva no correctiva. Utilice técnicas adecuadas para levantar objetos pesados. No nos hacemos responsables por lesiones durante su desempeño en cualquier trabajo físico. Manténgase fuera del alcance de los niños.

CAUTION

The belt is meant for prevention, not correction. Use proper techniques when lifting heavy objects. We are not responsible for any injuries sustained during physical work. Keep out of reach of children.

Care instructions: Hand wash with cold water and detergent. Do not use bleach or a dryer. Wash separately. Lay flat to dry in the shade. Do not wring or iron.